Single/Multi Drug Rapid Test (Urine)

(DE)	Produktinformation und Gebrauchsanleitung	2	(CZ)	Informace o produktu a návod k použití	16
(EN)	Product information and instructions for use	4	(FI)	Tuotetieto ja käyttöohje	18
(FR)	Informations sur le produit et instructions d'utilisation	6	(SE)	Produktinformation och bruksanvisning	20
(ES)	Información del producto e instrucciones de uso	8	(DK)	Produktinformation og brugervejledningen	22
(IT)	Informazioni ed istruzioni per l'uso	10	(NL)	Productinformatie en instructies voor gebruik	24
(PL)	Informacje o produkcie i Sposób użycia	12	(NO)	Produktinformasjon og bruksanvisning	26
(PT)	Informação sobre o produto e instruções de utilização	14		Symbols	28

Single/Multi Drug Rapid Test (Urine)

DE

1. LIEBER KUNDE

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EN

1. INTRODUCTION

Please read these notes carefully, because it contains important information on what should be considered when performing the test. Familiarise yourself with these instructions before using the contents of the package.

2. INTENDED USE

This rapid test is for the detection of drugs and their metabolites in urine.

3. WARNINGS AND PRECAUTIONS


- For single use only.
- Store tests between 2-30°C. Do not freeze.
- Do not use tests beyond the expiry date.
- Do not use tests with damaged pouches.
- Once removed from its foil pouch, the test should be used immediately.
- Do not touch the sample area or the reaction fields/pads.
- Do not touch the sample pad or the reaction fields/pads.
- Urine samples can be infectious. Tests should therefore be disposed of in household waste and hands washed thoroughly.

4. TEST PROCEDURE


Refrigerated tests and urine samples should be brought to room temperature (15-30°C).

1. Remove the test from the foil pouch. Remove the protective cap for multi-dip tests.
2. **A) DIP TEST:** Immerse the test strip(s) into the urine sample for **15-30 seconds** until the corresponding mark (MAX) is reached. The liquid must not exceed the MAX mark.
B) CASSETTE TEST: Transfer **3 drops of urine** into each sample well of the test cassette. Ensure that the urine does not come into contact with other areas of the test.
3. Place the test on a clean and flat surface. For multi-dip tests, replace the protective cap on the test cassette. Start the timer.
4. Read the results of the drug tests **after 5 minutes**. **Do not read the results after more than 8 minutes** (after no more than 10 minutes in the case of EtG).

A) DIP TEST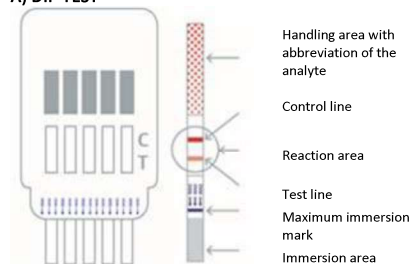

B) CASSETTE TEST

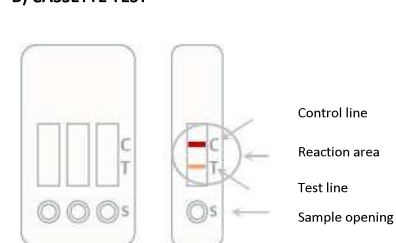5. INTERPRETATION OF THE RESULTS

DRUG TESTS

The test must be analysed under well-lit conditions so that weak colour lines can be detected. The reaction field contains a test line region (T) for the drugs that are to be detected, as well as a control line region (C).

Negative

A drug-free urine sample or a urine sample with drug content below the detection limit (cut-off) of the test was analysed. A test line appears in test line region (T) or, in a multi test device, all lines appear next to the respective parameter abbreviations.

Positive

No line(s) appear(s) in the test line region (T). In the case of mixed drug consumption, multiple test lines may be absent in the multi test device.


Single/Multi Drug Rapid Test (Urine)

Invalid


No control line appears in the control line region (C). With the control line (C), the rapid tests have an integrated process control. The control line confirms that sufficient sample material has been added and the test was performed correctly. If no control line appears, the test is invalid and must be disposed of. Please review the procedure and repeat the test with a new test. If the problem persists, discontinue using the batch immediately and contact your distributor.

Multi test :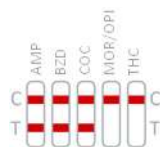

Negative for AMP+BZD+COC;
Positive for MOR/OPI+THC

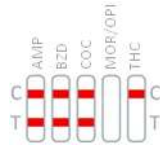Invalid for MOR/OPI;
Negative for AMP+BZD+COC;
Positive for THC

Multi-line test :

Negative for AMP+MOR/OPI+COC;
Positive for THC


Invalid for AMP+MOR/OPI;
Negative for COC;
Positive for THC

EtG-TESTS (for the detection of alcohol consumption)

The colour card is **only** necessary for the evaluation of EtG tests – all other drug tests can be evaluated without a colour card. For this reason, colour cards are only included with EtG tests.

Line Intensity Card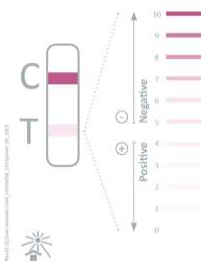

Example of the colour card.

- **Negative:** You will see a red line in the test zone. Compare this line with the example lines on the enclosed colour card. If the card colour intensity of the result is consistent with the colour of the sample lines for negative, then the result is considered negative.
- **Positive:** There will be no- or a very weak visible line in the test zone. Compare this with the example lines of the enclosed colour card. If the intensity of the test line is consistent with the colour of the sample lines for positive, then the result is considered positive.
- **Invalid:** No control line (C) appears. The test result should be disregarded. The test must be repeated with a new test device. If the problem persists, discontinue using the batch immediately and contact your distributor.

6. NOTE

The colour intensity of the C- and T-lines on the test membrane may vary between the individual parameters. Only if no T-line appears is the test positive.

Exception EtG Test:

The test evaluation is carried out using the enclosed colour card. For the interpretation of results, compare the colour of the line in the test line region (T) with the lines on the colour card.

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nal von minden GmbH